




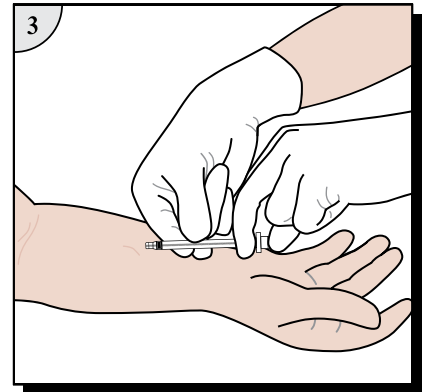

Suggested Technique For TB Skin Testing Using *1mL VanishPoint® Tuberculin Syringe*

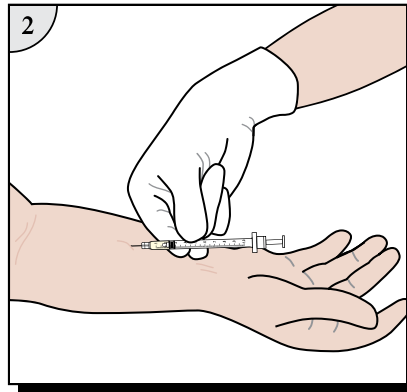

1. Prepare and give injection using aseptic technique according to institutional policy.
2. Using other hand, fully depress plunger to activate automatic needle retraction ***while needle is still in patient. Full dose is administered only when needle retraction is activated.***
3. Needle will automatically retract into syringe, preventing exposure to contaminated needle and rendering syringe non-reusable. In the event that needle retraction mechanism does not activate, discard syringe in an appropriate sharps container per protocol of institution. Do not recap contaminated needles.
4. Dispose of VanishPoint® syringe in an appropriate sharps container per protocol of institution.

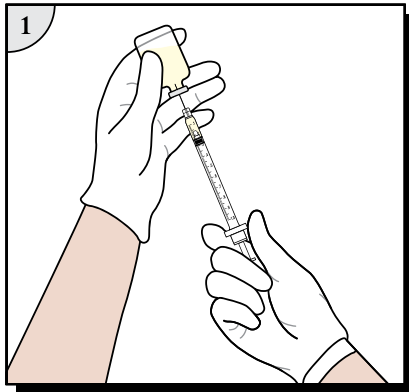