


## Needlestick prevention that meets your needs

- Pre-removal activation virtually eliminates exposure
- Single-handed, passive activation
- Hands remain behind the sharp
- Integrated safety mechanism
- Prevents needle removal
- Remains safe through disposal
- Easy to use
- Safe and effective for patients

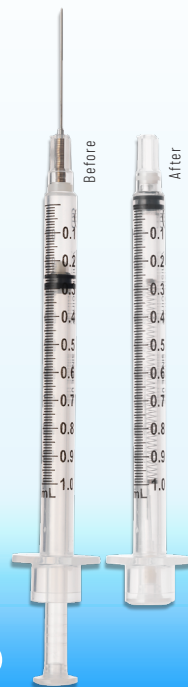

## SAFE FOR YOU AND YOUR PATIENTS

A study comparing the effectiveness of passive SED (safety-engineered devices) with that of active SED, concluded that passive SED (such as the VanishPoint® syringe) significantly reduced the needlestick injury rate.\*

### ATTACHED NEEDLE DESIGN:

- Reduces deadspace
- Maintains a closed system
- Prevents leakage and contamination
- Prevents unnecessary needle changes

\*A. J. Goris, et al., "Reducing Needlestick Injuries from Active Safety Devices A Passive Safety-Engineered Device Trial," *AOHP Journal*, vol. 34, no. 2 (Spring 2014) pp. 14-18.

## CATALOG

No.	Description	
<b>VanishPoint® U-100 insulin syringes</b> 100 per box/800 per case		
15391	0.5mL VanishPoint insulin syringe (half-unit)	30G x 3/16"
15371	0.5mL VanishPoint insulin syringe (half-unit)	30G x 5/16"
15291	0.5mL VanishPoint insulin syringe	30G x 3/16"
15271	0.5mL VanishPoint insulin syringe	30G x 5/16"
15221	0.5mL VanishPoint insulin syringe	30G x 1/2"
10291	1mL VanishPoint insulin syringe	30G x 3/16"
10271	1mL VanishPoint insulin syringe	30G x 5/16"
10281	1mL VanishPoint insulin syringe	29G x 5/16"
10211	1mL VanishPoint insulin syringe	29G x 1/2"
<b>1mL VanishPoint® syringes</b> 100 per box/800 per case		
10141	1mL VanishPoint syringe	30G x 5/16" <b>NEW</b>
10131	1mL VanishPoint syringe	27G x 1/2"
10151	1mL VanishPoint syringe	25G x 5/8"
10161	1mL VanishPoint syringe	25G x 1"
<b>VanishPoint® Allergy Syringe</b> Tray 25 per tray/1000 per case		
10134	1mL VanishPoint allergy syringes	27G x 1/2"
10154	1mL VanishPoint allergy syringes	25G x 5/8"
<b>3mL VanishPoint® syringes</b> 100 per box/600 per case		
13021	3mL VanishPoint syringe	27G x 1 1/2"
10301	3mL VanishPoint syringe	25G x 5/8"
10391	3mL VanishPoint syringe	25G x 1"
13011	3mL VanishPoint syringe	25G x 1 1/2"
10311	3mL VanishPoint syringe	23G x 1"
10321	3mL VanishPoint syringe	23G x 1 1/2"
10331	3mL VanishPoint syringe	22G x 1"
10341	3mL VanishPoint syringe	22G x 1 1/2"
10351	3mL VanishPoint syringe	21G x 1"
10361	3mL VanishPoint syringe	21G x 1 1/2"
10371	3mL VanishPoint syringe	20G x 1"
10381	3mL VanishPoint syringe	20G x 1 1/2"
<b>5mL VanishPoint® syringes</b> 100 per box/600 per case		
<b>10mL VanishPoint® syringes</b> 100 per box/600 per case		

5mL and 10mL VanishPoint® syringes are available with a variety of needle lengths and gauges. For a complete list, please contact RTI or visit our website at [www.retractable.com](http://www.retractable.com)

To order these products,  
please contact  
[rtisales@retractable.com](mailto:rtisales@retractable.com)

511 Lobo Lane, Little Elm, Texas 75068-5295 USA  
Tel.: (972) 294-1770, Fax: (972) 294-4400  
Toll Free: 1-888-703-1010




**RETRACTABLE**  
TECHNOLOGIES, INC.®



**VanishPoint®**  
SYRINGES

**The NEW standard  
for Real Safety™**

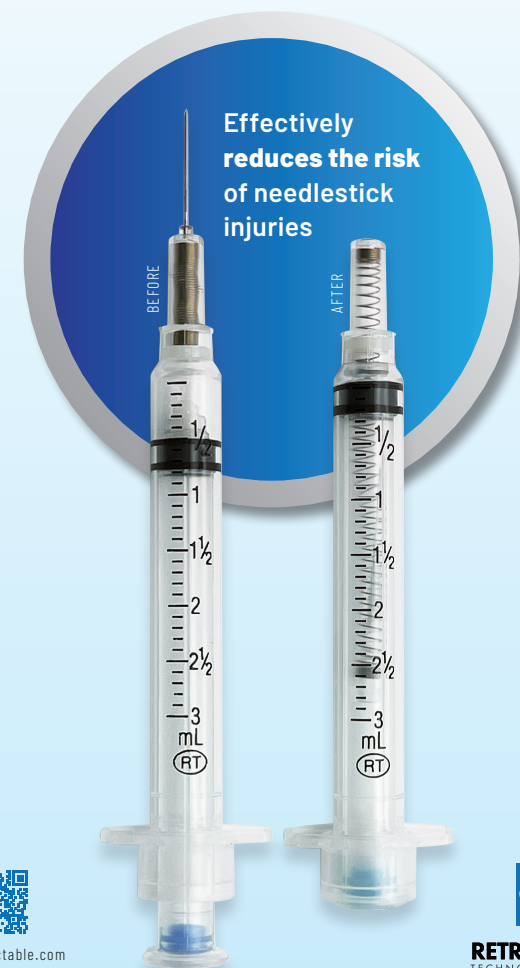

[www.retractable.com](http://www.retractable.com)



**RETRACTABLE**  
TECHNOLOGIES, INC.®



# VanishPoint®

# ADVANTAGE

## 1

Use standard procedure for drawing up medication.



## 2

Prepare and give injection using aseptic technique according to institutional policy.



## 3

While the needle is still in the patient, fully depress the plunger to activate retraction.



## 4

Dispose of syringe in an appropriate sharps container.




## Effective safety with automated retraction technology



Triple beveled, lubricated needle provides patient comfort

Clear, unobstructed Graduations allow for accurate dosing

**EASY, ONE-HANDED ACTIVATION**

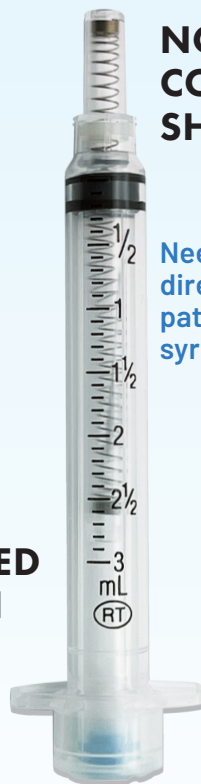

**NO CONTAMINATED SHARP EXPOSED**

Needle retracts directly from patient into syringe

**SAVES MONEY, TIME & LIVES**

**NEW DESIGN IMPROVED FOR YOU**

Same Retraction and Safety!



- **New Plunger Handle Design** for increased user comfort and control
- **Enhanced Bubble Removal** for quick and easy medication preparation

## DISPOSAL SAVINGS



VanishPoint® SYRINGES

Competitor

Activated VanishPoint® syringes require less disposal space than other safety syringes and prevent disposal-related injuries.