


Needlestick prevention that meets your needs

- Pre-removal activation virtually eliminates exposure
- Single-handed, passive activation
- Hands remain behind the sharp
- Integrated safety mechanism
- Prevents needle removal
- Remains safe through disposal
- Easy to use
- Safe and effective for patients

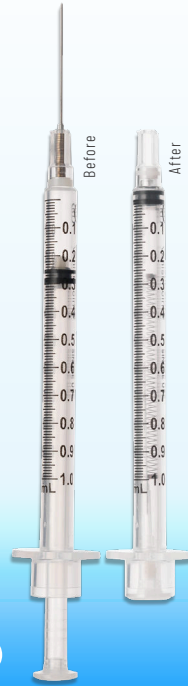

SAFE FOR YOU AND YOUR PATIENTS

A study comparing the effectiveness of passive SED (safety-engineered devices) with that of active SED, concluded that passive SED (such as the VanishPoint® syringe) significantly reduced the needlestick injury rate.*

ATTACHED NEEDLE DESIGN:

- Reduces deadspace
- Maintains a closed system
- Prevents leakage and contamination
- Prevents unnecessary needle changes

*A. J. Goris, et al., "Reducing Needlestick Injuries from Active Safety Devices A Passive Safety-Engineered, Device Trial," *ADHP Journal*, vol. 34, no. 2 (Spring 2014) pp. 14-18.

CATALOG

| No. | Description | |
|---|---|------------------------|
| VanishPoint® U-100 insulin syringes 100 per box/800 per case | | |
| 15391 | 0.5mL VanishPoint insulin syringe (half-unit) | 30G x 3/16" |
| 15371 | 0.5mL VanishPoint insulin syringe (half-unit) | 30G x 5/16" |
| 15291 | 0.5mL VanishPoint insulin syringe | 30G x 3/16" |
| 15271 | 0.5mL VanishPoint insulin syringe | 30G x 5/16" |
| 15221 | 0.5mL VanishPoint insulin syringe | 30G x 1/2" |
| 10291 | 1mL VanishPoint insulin syringe | 30G x 3/16" |
| 10271 | 1mL VanishPoint insulin syringe | 30G x 5/16" |
| 10281 | 1mL VanishPoint insulin syringe | 29G x 5/16" |
| 10211 | 1mL VanishPoint insulin syringe | 29G x 1/2" |
| 1mL VanishPoint® syringes 100 per box/800 per case | | |
| 10141 | 1mL VanishPoint syringe | 30G x 5/16" NEW |
| 10131 | 1mL VanishPoint syringe | 27G x 1/2" |
| 10151 | 1mL VanishPoint syringe | 25G x 5/8" |
| 10161 | 1mL VanishPoint syringe | 25G x 1" |
| VanishPoint® Allergy Syringe Tray 25 per tray/1000 per case | | |
| 10134 | 1mL VanishPoint allergy syringes | 27G x 1/2" |
| 10154 | 1mL VanishPoint allergy syringes | 25G x 5/8" |
| 3mL VanishPoint® syringes 100 per box/600 per case | | |
| 13021 | 3mL VanishPoint syringe | 27G x 1 1/2" |
| 10301 | 3mL VanishPoint syringe | 25G x 5/8" |
| 10391 | 3mL VanishPoint syringe | 25G x 1" |
| 13011 | 3mL VanishPoint syringe | 25G x 1 1/2" |
| 10311 | 3mL VanishPoint syringe | 23G x 1" |
| 10321 | 3mL VanishPoint syringe | 23G x 1 1/2" |
| 10331 | 3mL VanishPoint syringe | 22G x 1" |
| 10341 | 3mL VanishPoint syringe | 22G x 1 1/2" |
| 10351 | 3mL VanishPoint syringe | 21G x 1" |
| 10361 | 3mL VanishPoint syringe | 21G x 1 1/2" |
| 10371 | 3mL VanishPoint syringe | 20G x 1" |
| 10381 | 3mL VanishPoint syringe | 20G x 1 1/2" |
| 5mL VanishPoint® syringes 100 per box/600 per case | | |
| 10mL VanishPoint® syringes 100 per box/600 per case | | |

5mL and 10mL VanishPoint® syringes are available with a variety of needle lengths and gauges. For a complete list, please contact RTI or visit our website at www.retractable.com

To order these products, please contact

rtisales@retractable.com

511 Lobo Lane, Little Elm, Texas 75068-5295 USA
Tel.: (972) 294-1770, Fax: (972) 294-4400
Toll Free: 1-888-703-1010



RETRACTABLE
TECHNOLOGIES, INC.®



www.retractable.com



VanishPoint®
SYRINGES

The **NEW** standard
for **Real Safety™**



RETRACTABLE
TECHNOLOGIES, INC.®



VanishPoint®

ADVANTAGE

1

Use standard procedure for drawing up medication.



2

Prepare and give injection using aseptic technique according to institutional policy.



3

While the needle is still in the patient, fully depress the plunger to activate retraction.



4

Dispose of syringe in an appropriate sharps container.



Effective safety with automated retraction technology



Triple beveled, lubricated needle provides patient comfort

Clear, unobstructed Graduations allow for accurate dosing

EASY, ONE-HANDED ACTIVATION



NO CONTAMINATED SHARP EXPOSED

Needle retracts directly from patient into syringe

SAVES MONEY, TIME & LIVES

NEW DESIGN IMPROVED FOR YOU

Same Retraction and Safety!



- **New Plunger Handle Design** for increased user comfort and control
- **Enhanced Bubble Removal** for quick and easy medication preparation

DISPOSAL SAVINGS



VanishPoint® SYRINGES

Competitor

Activated VanishPoint® syringes require less disposal space than other safety syringes and prevent disposal-related injuries.