





# Blood Collection Set

## Product Usage Information:

- Select and prepare venipuncture site using aseptic technique according to institutional policy.
  - For blood collection using a tube holder, thread the luer adapter into a tube holder, while ensuring winged hub remains tightly connected to adapter.
  - For blood collection using a syringe, remove luer adapter or luer cap and attach syringe. Dispose of luer adapter in an appropriate sharps container per protocol of institution.
  - Holding device using the finger grips, carefully remove needle cover using a twisting motion. Do not apply pressure to the retraction trigger, indicated by the color-coded arrow (◀), when removing needle cover or when performing venipuncture.
1. Perform venipuncture, observing for blood return in device. Collect blood specimen according to institutional policy.
  2. Once blood collection is complete and the **needle is still in the patient's vein**, firmly depress retraction trigger indicated by the color-coded arrow (◀).
  3. The needle will automatically retract from the patient's vein into the device. In the event that needle retraction mechanism does not activate, discard blood collection set in an appropriate sharps container per protocol of institution. Do not recap contaminated needles.
  4. Dispose of blood collection set and tube holder (or syringe) in an appropriate sharps container per protocol of institution. Do not remove tube holder from blood collection set. For blood collection using a syringe, transfer blood to appropriate specimen container per institutional protocol.

## Precautions:

- Single use only. Reuse of this device may result in exposure to bloodborne pathogens, including Hepatitis B virus (HBV), Hepatitis C virus (HCV), and human immunodeficiency virus (HIV).
- Do not use if product or package is damaged.
- Not made with natural rubber latex, DEHP or PVC.
- The VanishPoint<sup>®</sup> blood collection set can be used for blood collection for diagnostic purposes or for short term IV administration, up to 2 hours.
- U.S. Federal Law restricts this device to sale by or on the order of a physician.

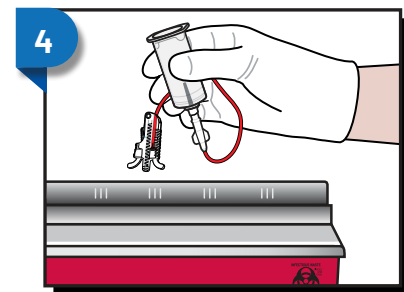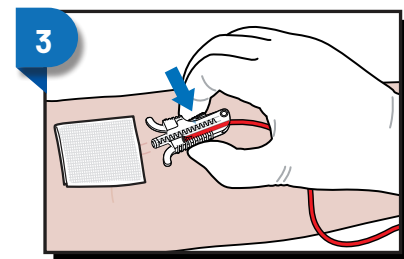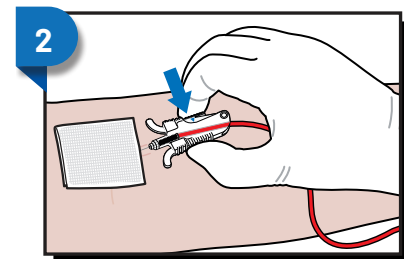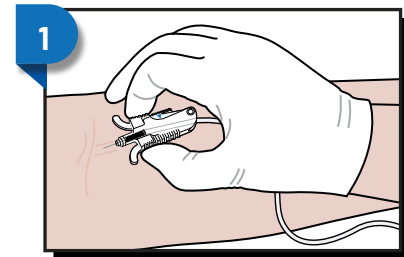

To order these products,  
please contact  
[rtisales@retractable.com](mailto:rtisales@retractable.com)



[www.retractable.com](http://www.retractable.com)



**RETRACTABLE**  
TECHNOLOGIES, INC.™

511 Lobo Lane, Little Elm, Texas, 75068-5295, USA  
Tel: (972) 294-1770, Fax: (972) 294-4400, Toll Free: 1-888-703-1010