


Suggested Technique For TB Skin Testing Using 1mL VanishPoint[®] Tuberculin Syringe

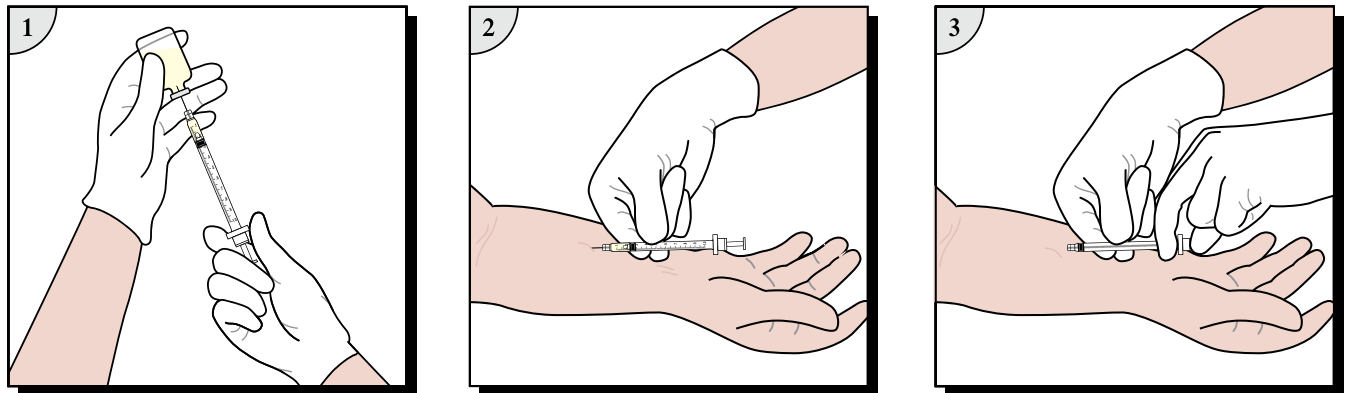

1. Prepare and give injection using aseptic technique according to institutional policy.
2. Using other hand, fully depress plunger to activate automatic needle retraction **while needle is still in patient. Full dose is administered only when needle retraction is activated.**
3. Needle will automatically retract into syringe, preventing exposure to contaminated needle and rendering syringe non-reusable. In the event that needle retraction mechanism does not activate, discard syringe in an appropriate sharps container per protocol of institution. Do not recap contaminated needles.
4. Dispose of VanishPoint[®] syringe in an appropriate sharps container per protocol of institution.



www.retractable.com

To order these products,
please contact
rtisales@retractable.com



RETRACTABLE
TECHNOLOGIES, INC.

511 Lobo Lane • Little Elm Texas • 75068-5295 • USA
Tel: (972) 294-1770 • Fax: (972) 294-4400 • Toll Free: 1-888-703-1010