

Read instruction leaflet found inside the box of VanishPoint syringes before use. VanishPoint syringes are available in different syringe and needle sizes. Your healthcare provider or pharmacist can help you choose the right size for your medication dose. Always follow the manufacturers' instructions for medication storage, preparation and administration.

1. GET READY


Wash your hands and gather all necessary supplies:

- Medication vial • Alcohol swabs • Syringe
- Disposal container

If necessary, remove cap covering medication vial. Clean the rubber stopper of the vial using an alcohol swab.

Choose and prepare injection site as advised by your healthcare provider.


2. PREPARE SYRINGE

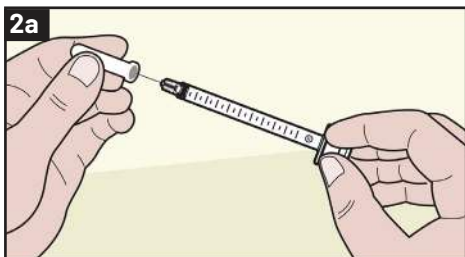


Peel open the VanishPoint syringe package at the tabs marked "PEEL THIS END".

2a Carefully remove the needle cap.

2b Pull back the plunger handle of the syringe, filling the syringe with air, until the leading edge of the black plunger seal is at the same number of milliliters (mL) required for your medication dose.

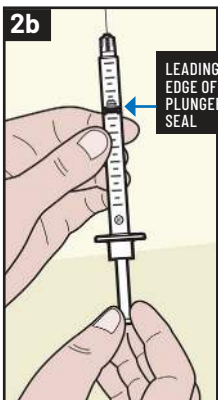


3. DRAW MEDICATION INTO SYRINGE

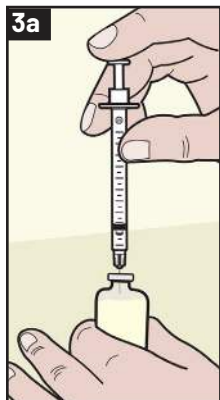

3a Insert the needle through the center of the rubber stopper of the vial. Take care not to bend the needle during preparation.

Depress the plunger handle, stopping at the point of resistance, to inject air into the vial of medication. In the event the plunger handle is fully depressed, resulting in the retraction of the needle, properly dispose of the retracted syringe and get a new syringe. Re-swab the rubber stopper of the vial with alcohol, insert the needle through the rubber stopper and resume with step 3b. It is not necessary to add any more air to the vial.

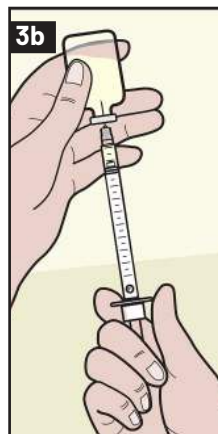

3b Turn the medication vial upside down, leaving the needle in place. The needle tip should be covered by medication.

Pull back slowly on the plunger handle to draw medication into the syringe to the graduation mark for your dose.

Check the syringe for any large bubbles. If any large bubbles are present, gently tap the side of the syringe barrel causing bubbles to rise to the needle end of the syringe. Carefully push the plunger forward to expel any large

bubbles. Withdraw additional medication into syringe, if necessary.

Check the syringe to make sure the leading edge of the black plunger seal is at the correct graduation mark for your dose of medication. Remove the needle from the vial.

4. GIVE INJECTION

4a At the prepared injection site, give the injection.

Always follow your healthcare provider's advice regarding injection technique.



4b Push the plunger handle. **While the needle is still in the skin**, fully depress the plunger handle, pushing past the slight resistance point. **The needle will automatically retract into the barrel of the syringe. The full dose of medication is injected when plunger handle is fully depressed** and the needle retraction mechanism is activated.

Dispose of the used syringe in an appropriate sharps disposal container.



To order these products, please contact
rtisales@retractable.com

511 Lobo Lane • Little Elm
Texas • 75068-5295 • USA
Tel: (972) 294-1770
Fax: (972) 294-4400
Toll Free: 1-888-703-1010



RETRACTABLE
TECHNOLOGIES, INC.[®]