



VanishPoint®



SAVES MONEY,
TIME & LIVES

SUGGESTED TECHNIQUE FOR RECONSTITUTING MEDICATION USING A VANISHPOINT® SYRINGE

Always follow:

- your facility's **protocol regarding medication reconstitution and preparation**,
- **aseptic technique** when drawing up any medication, and
- the **medication manufacturer's specific instructions** for the diluent type, volume, and mixing.

1. Insert VanishPoint needle into the diluent vial and draw up diluent dose per medication instructions.
2. Remove the needle from the vial and pull the plunger back an additional 0.1 - 0.2 mL, creating an air cushion between the diluent and the end of the syringe fluid chamber.
3. Insert the syringe needle into the vial of medication to be reconstituted, allowing the air to rise to the top.
4. Expel the diluent into the medication vial to be reconstituted, **without fully depressing the plunger**. The air cushion will allow all of the diluent to be expelled from the syringe without activating the retraction mechanism.
5. Mix reconstituted medication per medication instructions. The medication is now ready to be drawn into the VanishPoint syringe for injection.

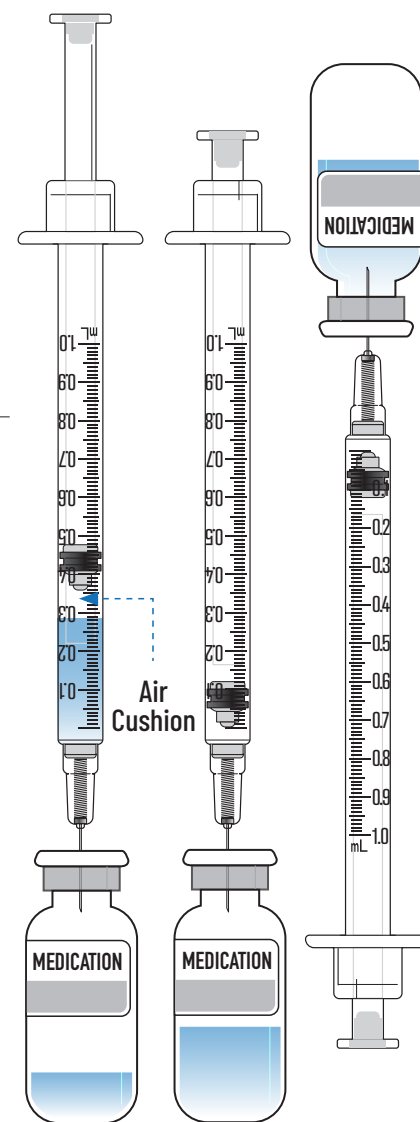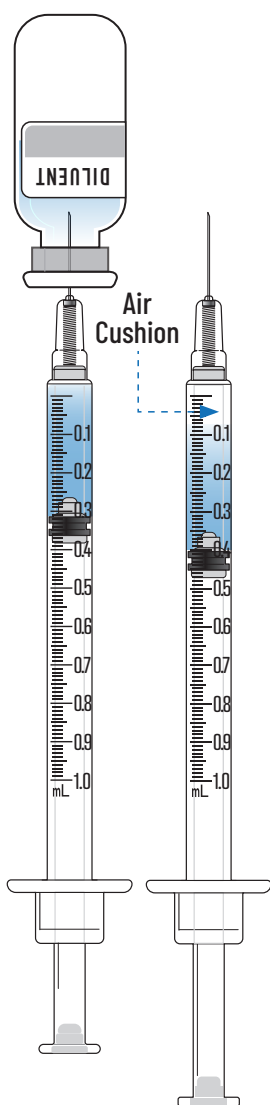

1

2

3

4

5

